BISCO **CE 0459**

CORE-FLO™ DC Dual-Cured Flowable Core Build-Up Composite

Instructions for Use

BISCO, Inc.
1100 W. Irving Park Road
Schamburg, IL 60193
U.S.A.
1-847-534-6000
1-800-247-3368

IN-1769R
Rev. 5/19

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed dentist.

CORE-FLO™™ DC Dual-Cured Flowable Core Build-Up Composite

GENERAL INFORMATION

CORE-FLO DC is a flowable dual-cure, fluoride-containing, radiopaque core build-up composite. **CORE-FLO DC** is specifically designed to cement posts and build-up the core structure in one easy application. **CORE-FLO DC** is also an ideal dentin replacement material. **CORE-FLO DC** comes in an auto-mixing cartridge, dual-syringe and is dispensed with the DUAL-SYRINGE DISPENSER. **CORE-FLO DC** is intended for use with a bonding agent that is compatible with dual-cured materials, such as BISCO adhesives.

Indications for use of **CORE-FLO DC** are:

- Core build-up material
- Cementation of posts or pins
- Dentin replacement material

Warnings:

- Contamination by saliva, blood, or water should be avoided during these procedures. When using dental adhesives, contamination will compromise dentin bonding and may result in decreased long-term use.
- A rubber dam is recommended for all indications where contamination may be possible.

Cautions:

- Cross-contamination: Do not use the same instrument for items that are designed for one time use. Dispose of used or contaminated tips. Do not clean, disinfect or reuse.
- Use of commonly available hygienic protective covering to avoid any contamination of syringes during treatment is recommended. If syringe becomes contaminated, discard. Do not clean or disinfect.

Precautions:

- Avoid contact with the skin; unpolymerized (meth)acrylate resins may cause skin sensitization in susceptible persons. In case of contact, wash skin with soap and water.
- Do not polymerize or cure the dual-syringe until the material should be kept in place until the next application. Use commonly available hygienic protective covering to avoid any contamination of syringes during treatment is recommended. If used intraorally, wipe mixing tip with disinfectant prior to storage.
- Do not use the dual-syringe for intraoral use. The dual-syringe is not intended for intraoral use. The dual-syringe is not intended for intraoral use. The dual-syringe is not intended for intraoral use.

Isolate the core build-up for taking impressions or fabricating temporary crownbridges (it may be placed directly on the preparation). Do not use the dual-syringe for intraoral use. The dual-syringe is not intended for intraoral use. The dual-syringe is not intended for intraoral use.

- Isolate the core build-up for taking impressions or fabricating temporary crownbridges (it may be placed directly on the preparation). Do not use the dual-syringe for intraoral use. The dual-syringe is not intended for intraoral use. The dual-syringe is not intended for intraoral use.
- Isolate the core build-up for taking impressions or fabricating temporary crownbridges (it may be placed directly on the preparation). Do not use the dual-syringe for intraoral use. The dual-syringe is not intended for intraoral use. The dual-syringe is not intended for intraoral use.

TECHNICAL INFORMATION
At room temperature (20°C/68°F - 25°C/77°F):
Working Time: Minimum 2 minutes
Setting Time: Maximum 7 minutes
Working time and Setting time may vary based on storage conditions, temperature, humidity, etc.

INSTRUCTIONS FOR USE

DUAL-SYRINGE INSTRUCTIONS FOR USE

Loading the Dual-Syringe into the DUAL-SYRINGE DISPENSER

- Position the Dual-Syringe dispenser's plunger to accept a Dual-Syringe cartridge by pushing up on the release lever located on the rear of the DUAL-SYRINGE DISPENSER. While the plunger releases, the Dual-Syringe cartridge will slide into the DUAL-SYRINGE DISPENSER.
- Push the Dual-Syringe cartridge into the DUAL-SYRINGE DISPENSER until one way. When viewed from the side, the cartridge will be flush with the front of the DUAL-SYRINGE DISPENSER. Slide the cartridge into the DUAL-SYRINGE DISPENSER until you hear a click. The click ensures that the cartridge is fully seated and locked into place.

Remove the Dual-Syringe cartridge cap and attaching the mixing tip

- To remove the Dual-Syringe cap, turn 90-degrees counterclockwise and pull it off.
- Before placing the Mixing Tip into the Dual-Syringe cartridge, use gentle steady pressure to the handle of the Dual-Syringe dispenser to pull the plunger and forth free the Dual-Syringe cartridge from both barrels of the Dual-Syringe cartridge and wipe onto a mixing pad. This is to ensure a homogeneous mixture of material.
- Select either the Root Canal Tip or Intra-Oral Tip and attach it to the Mixing Tip.
- Use the Mixing Tip to mix the material in the Dual-Syringe cartridge. The Mixing Tip is designed to mix the material in the Dual-Syringe cartridge. The Mixing Tip is designed to mix the material in the Dual-Syringe cartridge. The Mixing Tip is designed to mix the material in the Dual-Syringe cartridge.

Removal

- To remove the Dual-Syringe from the Dispenser, push the plunger release lever up and fully retract the Dual-Syringe. The Dual-Syringe will slide out of the Dispenser. It should slide freely, otherwise lift the release lever once again, pull back on the plunger and make certain that it is fully retracted.

1. CORE BUILD-UP USING CORE-FLO™ DC

- Isolate the tooth with a rubber dam.
- Apply an adhesive according to the manufacturer's instructions; we recommend the use of any BISCO adhesive.
- Follow the DUAL-SYRINGE INSTRUCTIONS FOR USE for loading of the Dual-Syringe for dispensing **CORE-FLO DC**.
- a. If using a matrix band, place a matrix band around the prepared tooth. Place the end of the intra-oral tip directly on to the preparation and dispense **CORE-FLO DC** light cure for 40 seconds.
- If using a core form, place the core form and dispense **CORE-FLO DC** light cure for 40 seconds. Place the loaded core form onto the prepared tooth and hold in place, light cure for 40 seconds.
- Prepare and finish the core in the usual manner using diamond and carbide burs.

2. POST CEMENTATION AND CORE BUILD-UP TECHNIQUE USING CORE-FLO™ DC

- Isolate the tooth with a rubber dam.
- Apply an adhesive according to the manufacturer's instructions; we recommend the use of any BISCO adhesive.
- Follow the DUAL-SYRINGE INSTRUCTIONS FOR USE for loading of the Dual-Syringe for dispensing **CORE-FLO DC**.
- Use the root canal mixing tip for the core build-up. Start at the apical end of the canal and withdraw the tip slowly while keeping the tip in the cement to minimize voids.
- Use the apical end of the post with **CORE-FLO DC** around the post to the desired height and shape. Remove the post and light cure for 40 seconds by placing the light on the desired height and shape.
- Dispense the desired amount of material by using gentle steady pressure. (See "TECHNICAL INFORMATION" Section for Working and Setting Time information.)
- Prepare and finish the core in the usual manner using diamond and carbide burs.

3. DENTIN REPLACEMENT MATERIAL

- Isolate tooth and prepare in a conservative manner. Rinse and dry.
- Apply an adhesive according to the manufacturer's instructions; we recommend the use of any BISCO adhesive.
- Follow the DUAL-SYRINGE INSTRUCTIONS FOR USE for loading of the Dual-Syringe for dispensing **CORE-FLO DC**.
- Express **CORE-FLO DC** only to the level of the dentin/enamel junction.
- Isolate the tooth with a rubber dam. Use a light-cure composite, building incrementally until cavity preparation is filled. Light cure according to manufacturer's instructions.
- Check the occlusion and continue with finishing and polishing procedures.

HYGIENE: To prevent polymerization or clumping, the dual-syringe mixing tip should be kept in place until the next application. Use of commonly available hygienic protective covering to avoid any contamination of syringes during treatment is recommended. If used intraorally, wipe mixing tip with disinfectant prior to storage.

DISINFECTION AND STERILIZATION: Disinfection/Method: Clean the dispenser with an ethanol solution or an impregnated dental wipe. Sterilization/Method: The dispenser may be sterilized in an autoclave at a temperature not to exceed 132°C for 10 minutes.

DISPOSAL: Refer to community provisions relating to waste. In their absence, refer to national or regional provisions relating to waste.

STORAGE: Store kit at room temperature (20°C/68°F - 25°C/77°F). See individual component labels for specific expiration dates.

WARNINGS: BISCO, Inc. recognizes its responsibility to replace products if proven to be defective. BISCO, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

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