BISCO ETCHANTS

Semi-Gel

Phosphoric Acid Etchant

UNI-ETCH® (32%)  0459 UNI-ETCH® (32%) W/BAC

ETCH-37™ (37%)  0459 ETCH-37™ (37%) W/BAC

Instructions for Use

 BISCO, Inc.
1100 W. Irving Park Road
Schaumburg, IL 60193
U.S.A.
1-800-247-3368
1-847-534-6000
www.bisco.com

 EC REP

BISICO France
208, allée de la Coudoulette
13680 Lançon de Provence
France
33-4-90-42-92-92

IN-039R21
Rev. 4/21

Caution: U.S. Federal Law restricts this device to sale by or on the order of a licensed Dentist.

en ETCHANTS
UNI-ETCH® (32%) UNI-ETCH™ (32%) W/BAC
ETCH-37™ (37%) ETCH-37™ (37%) W/BAC

Phosphoric Acid Etchant
GENERAL INFORMATION
BISCO ETCHANTS are phosphoric acid (H₃PO₄) intended for use on enamel, dentin, and as a cleaning agent for restorative substrates. BISCO ETCHANTS effectively remove the smear layer, etch and demineralize enamel and dentin, producing the necessary microretentive surface for successful bonding. The etchant rinses away cleanly and quickly, leaving no residue. BISCO ETCHANTS are also available with benzalkonium chloride (BAC), an antimicrobial agent. Bismuth citrate is the development of residual bacterial inhibition zones of up to 7 mm, when challenged (in-vitro) with Actinomyces viscosus.
NOTE: The inclusion of benzalkonium chloride has not been shown to correlate with a reduction in secondary decay in oral. In-vitro clinical studies of the bacteria effects of benzalkonium chloride on patients before or caries have not been performed.

Intended user: licensed dentist.

Indications For Use:
1. Etching dentin and enamel
2. Selective enamel etching
3. Cleaning agent on dental restorative materials

Warnings:
• The phosphoric acid in the etchant is a severe eye and skin irritant. Injury may result if the etchant is allowed to remain on the skin or mucosa for extended periods of time. In case of contact with other tissues, rinse immediately with plenty of water for several minutes.
• If accidentally splashed into the eye, flush with copious amounts of water and seek medical attention immediately.

Cautions:
Cross-contamination:
• Product may contain items that are designed for one time use. Dispose of used or contaminated tips. Do not clean, disinfect or reuse.
• Use of commonly available hygienic protective covering to avoid any contamination of syringes during treatment is recommended. If syringe becomes contaminated, discard. Do not clean or disinfect.

Precautions:
• Keep away from direct sunlight. Prolonged exposure to sunlight could cause a discoloration of the etchant; this will not compromise the efficacy of the product.
• Test the application of Etchant on a mixing pad or a glass slab, before use on patients so as to become familiar with the pressure required to dispense the Etchant from the tip. Never use intrinsically before checking flow from tip. Never use force on plunger to start flow intrinsically, this could result in a rapid, uncontrolled discharge of material. If resistance to flow is experienced, DO NOT proceed. Replace with new tip and verify flow before use on patient.
• Proprietary pastes containing oil or fluoride should not be used as these additives may interfere with etching.
• See individual component labels for specific expiration dates.
• Safety data sheet available on request.
• Safety data sheet available at www.bisco.com.
• Instructions for Use available upon request.
• Instructions for Use available at www.bisco.com.

INSTRUCTIONS FOR USE

- Etching dentin and enamel**
 - Remove white cap from etchant syringe.
 - Attach the light blue 25 gauge tip.
 - Verify flow of etch before intraoral application. Dispense a small portion of etch onto a pad or gauze to familiarize yourself with its viscosity.
 - If surface to be etched is not freshly cut, clean with a pumice-water slurry. Proprietary pastes containing oil or fluoride should not be used as these additives may interfere with etching. Rinse well.
 - Isolate the preparation.
 - Place etchant directly onto the dentin and enamel surfaces to be etched. Leave etchant in place for 15 seconds.
 - Thoroughly rinse with water for 5 seconds with suction/1 selectively etching enamel, suction prior to rinsing to avoid etchant flowing on to the dentin.
 - Immediately proceed with the bonding procedure.
 - Discard all single use components.
- Selective enamel etching**
 - Remove white cap from etchant syringe.
 - Attach the light blue 25 gauge tip.
 - Verify flow of etch before intraoral application. Dispense a small portion of gel upon a pad or gauze to familiarize yourself with its viscosity.
 - If surface to be etched is not freshly cut, clean with a pumice-water slurry. Proprietary pastes containing oil or fluoride should not be used as these additives may interfere with etching. Rinse well.
 - Isolate the preparation.
 - Place etchant directly onto the surfaces to be etched. Leave etchant in place for 15 seconds.
 - Thoroughly rinse with water for 5 seconds with suction/1 selectively etching enamel, suction prior to rinsing to avoid etchant flowing on to the dentin.
 - Immediately proceed with the bonding procedure.
 - Discard all single use components.
- Cleaning agent on dental restorative materials**
 - BISCO ETCHANTS may be used on restorative substrates to effectively remove organic contaminants. Apply etchant to the surface for 30 seconds and rinse thoroughly with water.
NOTE: Unprimed zirconia surfaces should NOT be cleaned with phosphoric acid. Use a restoration cleaner such as ZiClean to remove contaminants.
 - HYGIENE: Re-cap syringes with the luer lock cap.

DISPOSAL: Refer to community provisions relating to waste. In their absence, refer to national or regional provisions relating to waste.

STORAGE: Store at room temperature (20°C/68°F - 25°C/77°F). Store away from direct sunlight.

WARRANTY: BISCO, Inc. recognizes its responsibility to replace products if proven to be defective. BISCO, Inc. does not accept liability for any damages or loss, either direct or consequential, stemming from the use of or inability to use the products as described. Before using it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

* UNI-ETCH and ZiClean are registered trademarks of BISCO, Inc.

ETCH-37 is a trademark of BISCO, Inc.

NOTE: Report any serious incident to BISCO and the competent authority of the state in which the user and/or patient is established.

TO REORDER:
X-80611N - 125 Light Blue Disposable Tips (25 Gauge)
X-80608N - 50 Dark Blue Disposable Tips (22 Gauge)

1. Kanca, J. J. "Etchant Composition and Bond Strength to Dentin". Am J Dent 1993;6:162-164.
2. Dr. Daniel Chan, University of Texas Health Science Center at San Antonio Dental School. Residual Effect of 1 and 2% Benzalkonium Chloride Incorporated into an Etchant on the Susceptibility of Actinomyces viscosus T14V 1993.

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3. Reinigungssubstanz auf dentalen Restaurationsmaterialien.
1. BISCO ETCHANTS may be used on restorative substrates to effectively remove organic contaminants. Apply etchant to the surface for 30 seconds and rinse thoroughly with water.
NOTE: Unprimed zirconia surfaces should NOT be cleaned with phosphoric acid. Use a restoration cleaner such as ZiClean to remove contaminants.

HYGIENE: Re-cap syringes with the luer lock cap.

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