BISCO

CE 0459

Z-PRIME™ PLUS

Zirconia - Alumina - Metal Primer

Instructions for Use

BISCO, Inc.
1100 W. Irving Park Road
Schaumburg, IL 60193
U.S.A.
1-800-247-3368
1-847-534-6000
www.bisco.com

BISCO France
208, allée de la Coudoulette
13680 Lançon de Provence
France
33-4-90-42-92-92

Patent No. 8,785,518

BISCO France
208, allée de la Coudoulette
13680 Lançon de Provence
France
33-4-90-42-92-92

IN-178R12
Rev. 3/21

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed Dentist.

en Z-PRIME™ Plus
Zirconia - Alumina - Metal Primer

GENERAL INFORMATION

BISCO's **Z-PRIME Plus** is a single-component priming agent used to enhance adhesion between indirect restorative materials and composite resin cements. Due to its unique chemistry it can be used as a surface treatment for Zirconia and Alumina metal oxide ceramics as well as other types of metal alloys, composites and endodontic posts. **Z-PRIME Plus** has been specifically formulated to provide high bond strengths in any curing mode.

Indications for use:
BISCO's **Z-PRIME Plus** is intended for use as a surface treatment for the following restorations:

1. Zirconia
2. Alumina
3. Metal alloy
4. Composite
5. Endodontic Post
6. Intraoral repair
7. Intraoral Intervention

Intended user: licensed dentist.

Warnings:

- Highly flammable.

Cautions:

- Cross-contamination: Product may contain items that are designed for one time use. Dispose of used or contaminated accessories. Do not clean, sterilize or reuse.

Precautions:

- **Z-PRIME Plus** is a light sensitive agent. Use **Z-PRIME Plus** immediately once it is dispensed in the mixing well or protect from ambient light.
- For intraoral repairs properly isolate the restorative area with a rubber dam.
- Avoid contact with the skin. unpolymerized (meth)acrylate resins may cause skin sensitization in susceptible persons. In case of contact, wash skin with soap and water.
- Safety data sheet available on request.
- Safety data sheet available at www.bisco.com.
- Instructions for Use available upon request.
- Instructions for Use available at www.bisco.com.

Restoration Preparation
Zirconia Restorations/Alumina Restorations, Metal/Alloy restorations including Porcelain Fused to Metal, Indirect Composite Restorations, Endodontic Post Restorations

NOTE: Make sure restoration has been properly sandblasted, or sandblast restoration according to manufacturer's instructions. Sandblasting should be performed prior to placement of **Z-PRIME Plus**.

1. Brush on 1-2 coats of **Z-PRIME Plus** inside the restoration. Wait for 30 seconds and air dry. **NOTE:** Surface of the prime zirconia will look like same before and after primer application.
2. Try-in the primer treated-restoration on the preparation for proper fit. Rinse the restoration thoroughly with water and air dry with air-free air.
3. **NOTE:** If the surface was not pre-treated with **Z-PRIME Plus**, decontaminate the surface with a cleaning agent (e.g. **ZiClean**). Rinse with water spray and air dry with 100% air. Then follow step 1.
4. Proceed with cementation using a luting cement (e.g. Duo-Link Universal™ or BioCem®).

For intraoral repairs
NOTE: Properly isolate the restorative area prior to intraoral repair.

Restoration of Zirconia/Alumina or Metal Restorations:

1. Roughen the surface using air abrasion, diamond point or bur. Rinse and air dry.
2. Apply 1-2 coats of **Z-PRIME Plus**. Dry with an air syringe for 3-5 seconds.
3. Complete the repair using a composite (e.g. **ELITE™**) and finish per manufacturer's instructions.

Repair of Porcelain-Fused-to-Zirconia/Alumina/Metal Restorations:

1. Bevel the porcelain as necessary.
2. Roughen the metal surface using air abrasion, diamond point or bur. Rinse and air dry.
3. Etch the porcelain surface with hydrofluoric acid (e.g. 9.5% PORCELAIN ETCHANT) according to the manufacturer's instructions.
4. Apply 1-2 coats of silane (e.g. **BIS-SILANE™** or PORCELAIN PRIMER) to the porcelain according to the manufacturer's instructions & dry with an air syringe to evaporate the solvent.
5. Apply 1-2 coats of **Z-PRIME Plus** to the exposed zirconia/alumina/metal & dry with an air syringe for 3-5 seconds.
6. Apply an opaquer (e.g. Bisco's DUAL-CURED OPAQUE) to exposed metal according to manufacturer's instructions if desired.
7. Complete the repair using a composite (e.g. **ELITE**) and finish per manufacturer's instructions.

Repair of Composite Restorations:

1. Bevel the margin.
2. Roughen surface using air abrasion, diamond point or bur. Rinse and air dry.
3. Apply 1-2 coats of **Z-PRIME Plus**. Dry with an air syringe for 3-5 seconds.
4. Complete the repair using a composite (e.g. **ELITE**) and finish per manufacturer's instructions.

STORAGE: Store **Z-PRIME Plus** at (2°C/36° - 25°C/77°F). See individual component labels for specific expiration dates.

DISPOSAL: Refer to community provisions relating to waste. In their absence, refer to national or regional provisions relating to waste.

WARNINGS: BISCO, Inc. recognizes its responsibility to replace products if proven to be defective. BISCO, Inc. does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Z-PRIME, Duo-Link Universal, ZiClean, ELITE and BIS-SILANE are trademarks of BISCO, Inc.
BioCem is a registered trademark of BISCO, Inc.
9.5% PORCELAIN ETCHANT, PORCELAIN PRIMER and DUAL-CURED OPAQUE are manufactured by BISCO, Inc.

NOTICE: Report any serious incident to BISCO and the competent authority of the EU member state in which the user and/or patient is established.

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