
BISCO

ZirClean®

Restoration Cleaner

Instructions for Use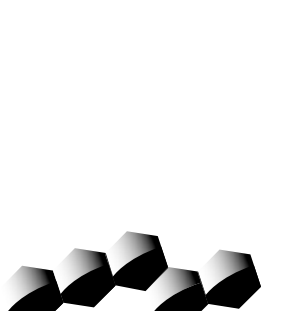

BISCO, Inc.
1100 W. Irving Park Road
Schamburg, IL 60193
U.S.A.
1-800-247-3368
1-847-534-6000
www.bisco.com

BISCO France
208, allée de la Coudelette
13680 Lançon de Provence
France
33-4-90-42-92-92

IN-225R10
Rev. 5/22

Caution: U.S. Federal law restricts this device to sale by or on the order of a Licensed Dentist.

en **ZirClean®**
Restoration Cleaner

GENERAL INFORMATION

ZirClean is a cleaning gel suitable for non-abrasive cleaning of the bonding surfaces of prosthetic restorations after intraoral try-in. Consequently, it helps to achieve reliable adhesive cementation results.

Intended user: licensed dentist.

Indications for use:
ZirClean is an intraoral cleaner of pre-treated, etched, zinciroal and metal restoration surfaces which have been contaminated during intraoral try-in.

Contraindications:

- If the stipulated working technique cannot be applied.
- If cleaning is performed intraorally.
- For the cleaning of restorations which have come in contact with silicone-based pastes.

Warnings:

- May cause chemical burns.
- Do not breathe in fumes.
- In case of contact with eyes, immediately with plenty of water and seek medical advice.
- Wear suitable protective clothing, gloves and eye/protection.
- DO NOT use **ZirClean** to etch exposed enamel or dentin surfaces.
- Injury may result if **ZirClean** is allowed to remain on the skin for extended periods of time. Contact with other body tissues causes, irreversibly, with a copious amount of water for several minutes.
- If resistance is found during dispensing, replace tip and re-apply. Use only with provided dispensing tips. Use retractor accessory items below.
- Do not allow any product to be ingested or swallowed.
- Re-cap syringe with the new Luer lock.
- This material and its container must be disposed of as hazardous waste. Follow disposal instructions.

Caution:

- Cross-contamination: Product may contain items that are designed for one-time use. Dispose of used or contaminated accessories. Do not clean, disinfect or reuse disposable accessories.
- Use of commonly available hygienic protective covering to avoid any contamination of syringes during restoration is recommended, if syringe becomes contaminated, follow disposal instructions. Do not clean or disinfect.

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INSTRUCTIONS FOR USE

NOTE: Independent of whether or not the restorations have been conditioned beforehand, they should be cleaned according to the following cleaning protocol.

Non-Abrasive Materials (Zirkon, Metall)

1. In (to) check occlusion and fit.
2. After try-in, thoroughly rinse the restoration with water spray and dry with oil-free air.
3. Remove cap of **ZirClean** securely attach tip and verify flow of material prior to application.
4. Cover all bonded surfaces of the restoration with a layer of **ZirClean**. Tips in glass use only, make sure to dispose of them properly before using material again.
5. Allow 20 seconds for the cleaning action of **ZirClean** to take effect, then thoroughly rinse with water spray and dry with oil-free air.
6. Sandblast internal surface of restoration (unless restoration has previously been sandblasted).
7. Prime the bonding surface of the restoration with a suitable primer (e.g.: Z-PRIME™ Plus) according to manufacturer's instructions.

Etchable Materials (Lithium Disilicate, Feldspathic Porcelain)

1. In (to) check occlusion and fit.
2. After try-in, thoroughly rinse the restoration with water spray and dry with oil-free air.
3. Remove cap of **ZirClean** securely attach tip and verify flow of material prior to application.
4. Cover all bonded surfaces of the restoration with a layer of **ZirClean**. Tips are single use only, make sure to dispose of them properly before using material again.
5. Allow 20 seconds for the cleaning action of **ZirClean** to take effect, then thoroughly rinse with water spray and dry with oil-free air.
6. Prime the bonding surface of the restoration with a suitable primer (e.g.: PORCELAIN PRIMER™, or Bio-Silane™**) according to manufacturer's instructions.

HYGIENE: Re-cap syringes with the Luer lock cap.

DIPSOAL: Report to community provisions relating to waste. In their absence, report to national or regional provisions relating to waste.

STORAGE: Store at room temperature (20°C/68°F - 25°C/77°F).

WARRANTY: BISCO, Inc. recognizes its responsibility to replace products if proven defective. BISCO, Inc. does not accept liability for any damages or consequential loss resulting from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

NOTICE: Report any serious incident to BISCO and the competent authority of the EU member state in which the user and/or patient is established.

TO REORDER:

X-8021N - 22 gauge blank dispensing tip (50 count)

*ZirClean is a registered trademark of BISCO, Inc.
*Z-PRIME and Bio-Silane are trademarks of BISCO, Inc.
*PORCELAIN PRIMER is manufactured by BISCO, Inc.
*Based on comparison to untreated samples

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